



Safety facial shield #1

Mounting instruction



1 Safety face shield #1 and components



1 x Face Shield



1 x Fastening Strap




2 x Rubber Band

2 Attach fastening strap to the face visor using hooks and cuts placed on both, top sides of the mask



3

Pull the rubber band through the hole in the bottom corner of the visor and attach it to the hook. Do the same on the opposite side.

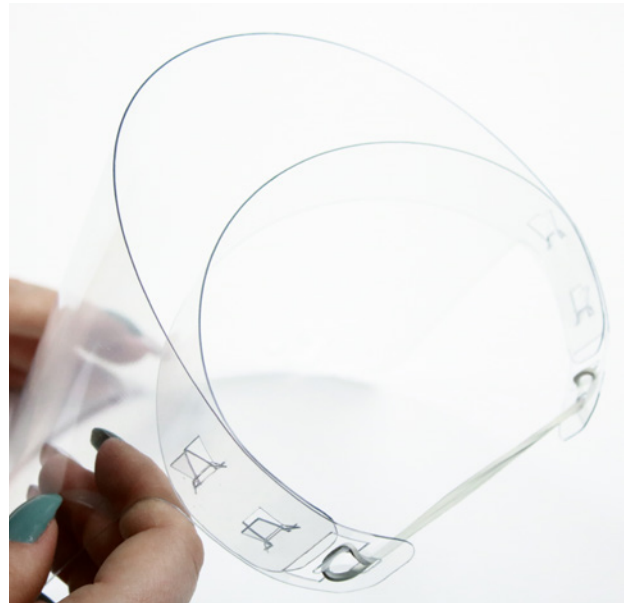

4

Pull the rubber band through the top corner of the mask and attach it to the hook. Do the same on the opposite side.



Unsterilised Product



Single-Use Product



Use at temperature of 5 – 30 °C